**activeblue**® 

CE

Gebrauchsanweisung	S. 2
Instructions for use	S. 18
Mode d'emploi	S. 34
Modo de empleo	S. 50
Modalità d'uso	S. 66

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hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EEC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of unit:	<b>activeblue</b>	
	REF 098-000-00	
Start with Unit No.:	0041 - 2011	
EEC guidelines:	2006/42/EC	Guideline for machine
	2006/95/EC	Electrical equipment used within certain voltage limits
	2004/108/EC	Electromagnetic compatibility
Applied unified standards:	EN ISO 12100-1:2003 EN ISO 12100-2:2003 EN ISO 14121:2007 EN 61010-1:2001 EN 61326-1:2006 EN 61326-2-2:2006	

Date and manufacturers signature: 05.07.2011  
Signatory:

-----  
- i.V. Dipl.Ing. (FH) K. Merkle -  
Production Manager Mechanic

Printing Date: 10.08.11

## Index

Introduction.....	20
User Notices.....	20
Symbols .....	21

## Instructions for use

1. Unpacking and installing .....	23
2. Presentation.....	23
3. Appliance Description .....	23
4. Connecting and switching on.....	23
5. Operation .....	24
5.1. Preparing the object.....	24
5.2. Cleaning fluid .....	25
5.3. Filling the cleaning beaker.....	25
5.4. Choose cleaning time and RPM.....	25
5.5. Starting, pausing, and stopping cleaning.....	26
5.6. Remove the cleaning object .....	26
5.7. Rinsing and refilling cleaning needles .....	26
5.7.1. Rinsing .....	26
5.7.2. Adding cleaning needles.....	27
5.8. Rinsing the beaker cover .....	27
6. Cleaning and Maintenance .....	27
6.1. Replacing cleaning needles .....	27
6.2. Cleaning beaker.....	28
6.3. Changing a fuse .....	28
7. Spare Parts .....	28
8. Accessories .....	28
9. Troubleshooting .....	29

## Information for the purchaser

A. Application.....	30
A.1 Correct use of the unit.....	30
A.2 Unapproved use .....	30
A.3 Ambient conditions.....	30
B. Hazards and warnings .....	31
C. Approved personnel.....	32
D. Prior to installation .....	32
E. Repair .....	32
F. Disposal instructions .....	32
F.1 Disposal instructions for countries in the EU .....	32
G. Technical data .....	32
H. Operating duration .....	33
I. Disclaimer.....	33
K. Guarantee .....	33
Spare Parts .....	83

# Introduction

## Dear Customer

Thank you for having chosen a quality product from Dentaaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at [www.dentaaurum.de](http://www.dentaaurum.de).

The **activeblue high-performance cleaning device REF 098-000-00** is a compact cleaning unit for all types of removable orthodontic appliances. The cleaning process is based on mechanical cleaning using magnetisable needles in a rotation process in conjunction with **activefluid cleaning liquid**. This special cleaning liquid, which was specially developed for cleaning orthodontic appliances, allows quick and effective cleaning of orthodontic appliances to remove stubborn plaque and calculus.

This appliance sets new standards of functionality, performance and ergonomics.

## Manufacturer

Dentaaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

**To guarantee long and trouble-free use please read the following instructions for use carefully and always follow the included safety notices.**

## User Notices

**Instruct the user according to the information in this manual on application, possible dangers, and operation of the activeblue high-performance cleaning device.**

This user information should be made available to the operator at all times.

**Further information about the areas of application and reliable use of the unit is available in the section "Information for the operator" at the end of these instructions for use.**

## Symbols

This instructions for use contains symbols also found on the appliance itself with the following meaning:

**Danger: Immediate risk of injury**

**High Voltage: Danger – high voltage**

**Caution: Ignoring this notice may cause damage to both appliance and object.**

**Notice: A general notice that improves and eases use.**

**For indoor use only.**

**Disconnect from mains supply before opening the appliance.**

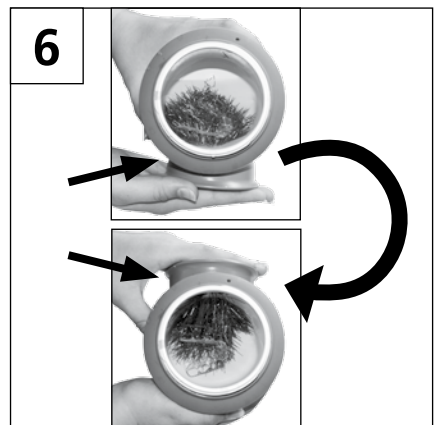
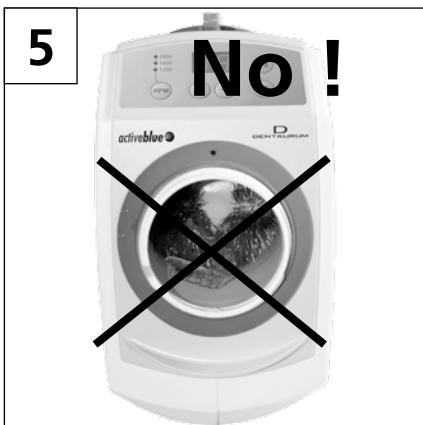
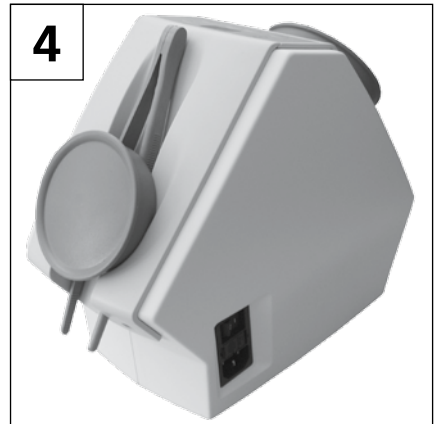
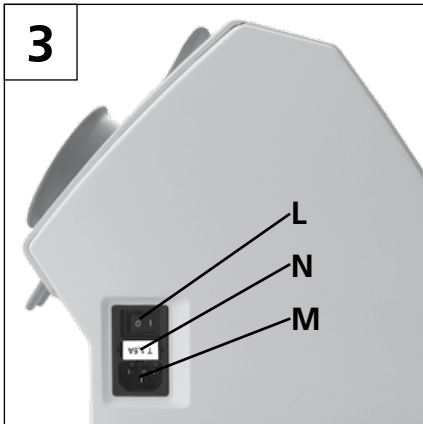
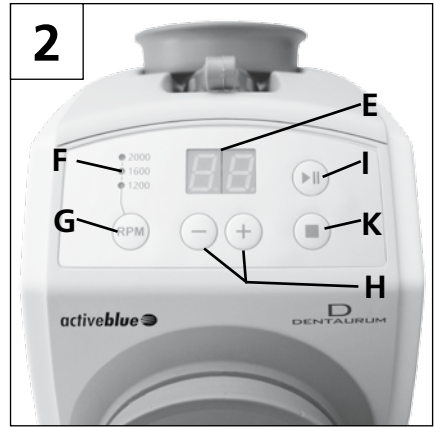
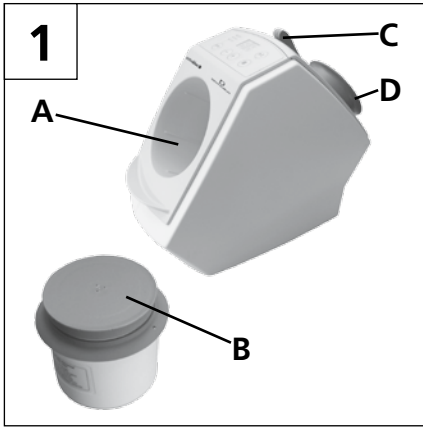
**The appliance complies with current EU directives.**

**Within the EU, this device is subject to the provisions of the WEEE directive.**

**Other symbols are explained when displayed.**

# 1. Unpacking and installing

Remove the appliance and all accessories from the packaging.

Place the appliance on a bench of comfortable height. Place the tweezers supplied with the unit and the retaining magnets in the holder on the back of the appliance (Fig. 4).

**Danger of injury through falling equipment! Remove the retaining magnets from the holder when transporting the appliance.**

## 2. Presentation

- 1 **activeblue high-performance cleaning device**
- 1 Cleaning beaker, with cover (for use in the cleaning device with **activefluid cleaning liquid**)
- 1 Cleaning needles (container, 75 g)
- 1 **activefluid cleaning liquid**, sample bottle (400 ml),
- 1 Tweezers
- 1 Retaining Magnet
- 1 Mains Cable (shockproof plug, 230 V)
- 1 Instructions for use

## 3. Appliance Description

(See Fig. 1, 2, and 3)

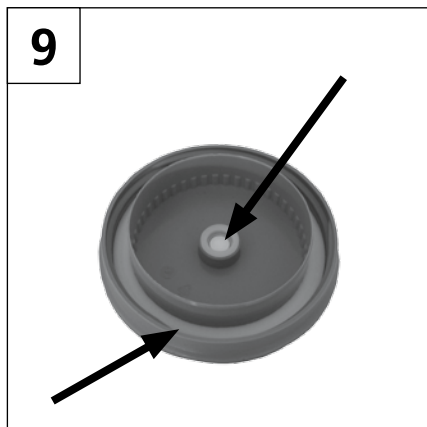
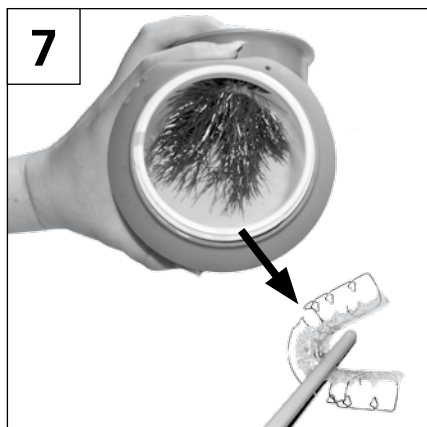
- (A) **activeblue high-performance cleaning device**
- (B) Cleaning beaker for needle cleaning using **activefluid cleaning liquid**
- (C) Tweezers
- (D) Retaining magnet
- (E) Cleaning time indicator
- (F) RPM indicator
- (G) RPM selection key
- (H) +/- buttons, change cleaning time
- (I) Start/ pause button
- (K) Stop button
- (L) Mains switch
- (M) Mains connection
- (N) Fuse

## 4. Connecting and switching on

**Ensure the mains supply and voltage marked on the nameplate is identical.**

- Connect the supplied power cable to the mains connection socket (M, Fig 3).
- Connect the power cable to the mains supply.
- Turn on the appliance at the mains switch (L, Fig. 3).

## 5. Operation

### 5.1. Preparing the object

- ! Inspect the object for unstable areas or damage before cleaning it in the appliance. These areas may be affected by chemical and mechanical stress during cleaning.
- ! Delicate areas and function elements may be damaged.
- i High-gloss areas may become dulled through cleaning.

### 5.2. Cleaning fluid

**activefluid cleaning liquid:** Special cleaner for use in the **activeblue high-performance cleaning device**. Ready-to-use solution for the removal of stubborn soils, bleach, calculus.

Physical and chemical properties: Acid cleaning liquid, pH=2,0.

### Please observe the processing order:

1. Clean using 200 ml **activefluid cleaning liquid** in the cleaning device for approx. 20 min.
2. Remove any residual stains by simple mechanical cleaning with a toothbrush under warm running water.
3. Rinse and disinfect the cleaned object thoroughly before transferring to the patient.

**The submerged object may cause bacterial contamination of the activefluid cleaning liquid. For this reason: Always use fresh activefluid cleaning liquid for every new cleaning sequence!**

### Information on disposal:

The used **activefluid cleaning liquid** can be disposed of in the waste water system. Please observe local and national regulations. Request information from your local authorities about proper disposal.

### Delivery and storage:

With the unit: 400 ml container (sample).

Reordering: 1000 ml container (REF 150-020-00)

- Store only in the closed original container between -5 °C / 23 °C and 40 °C / 104 °C.
- Keep cool, dark and in an upright position.

## 5.3. Filling the cleaning beaker

For use in the **activeblue high-performance cleaning device**.

- Fill the white cleaning beaker (B, Fig. 1) with:
  - 75 g cleaning needles (corresponding to one packaging unit),
  - 200 ml **activefluid cleaning liquid**, equivalent to filling the beaker to the MIN mark.
  - the object to be cleaned. Make sure the liquid level stays clear of the MAX mark.
- Close the beaker.
- Place the beaker in the appliance.

**Do not overfill the beaker.**

**Never use the appliance with the cleaning beaker open (Fig. 5).**

## 5.4. Choose cleaning time and RPM

- Adjust the cleaning time with the +/- buttons (H, Fig. 2) (range 0 to 30 min).
- Adjust RPM with the selector (G, Fig. 2). The indicator now shows the chosen RPM.

**The cleaning time in the appliance will depend on the nature of the selected RPM and the degree of soil and staining. It is advisable to check the results several times during cleaning if you have no previous experience with this appliance.**

**Never open the beaker while still inside the appliance (Fig. 5).**

Recommended RPM:

1200 min<sup>-1</sup>: Cleaning objects with delicate areas and function elements.

1600 min<sup>-1</sup>: Standard setting for orthodontic appliances.

2000 min<sup>-1</sup>: Cleaning objects with heavy contamination and stubborn plaque.

## 5.5. Starting, pausing, and stopping cleaning

### Start cleaning with the Start/ Pause button (I, Fig. 2).

- The appliance stops automatically when the set cleaning time has elapsed.
- The remaining time is displayed in the indicator during cleaning.
- Cleaning can be stopped and restarted by pressing the Start/ Pause button.
- Once cleaning is stopped, the remaining cleaning time can be adjusted.
- You can change the set RPM at any time during cleaning.
- Cleaning can be stopped at any time before the cleaning time has elapsed by pressing the Stop button.
- After the cleaning time has elapsed the indicator shows „00“.
- Stop cleaning altogether by pressing the Stop button (K, Fig. 2). The display shows the previously set cleaning time.

**To improve the cleaning performance, the sense of rotation will change every 30 s**

**With especially stubborn stains it may be necessary to alter to a longer running time (+ max. 10 min) and then short, manual cleaning, under running water, using a hard toothbrush.**

## 5.6. Remove the cleaning object

After cleaning:

- Remove the cleaning beaker and open the lid while holding over a washbasin or sink unit.

**Never open the beaker while still inside the appliance (Fig. 5).**

- Place the retaining magnet on the outer wall of the beaker such that the cleaning needles accumulate on the inner wall (Fig. 6).
- You can now remove the object with the enclosed pliers (Fig. 7).
- Before delivering the object to the patient, rinse it with a copious amount of water and disinfect.

**Carefully check the object for any adhering cleaning needles before delivering.**

## 5.7. Rinsing and refilling cleaning needles

**The cleaning needles can be used more than once.**

### 5.7.1. Rinsing

After each cleaning cycle, rinse the cleaning needles and prepare fresh **activefluid cleaning liquid**.

To do this:

- Hold the retainer magnet against the underside of the cleaning beaker (Fig. 8).
- Carefully pour out the **activefluid cleaning liquid**

**The magnet holds the cleaning needles in the beaker.**

- Rinse the beaker and cleaning needles with clean water.
- Hold the retainer magnet against the underside of the cleaning beaker and pour out the water.
- Once rinsed, the needles can be kept for longer periods of time in an open, well dried cleaning beaker.

### 5.7.2. Adding cleaning needles

Some cleaning needles will invariably be lost when cleaning the needles and the orthodontic appliances. This will reduce the cleaning performance. To bring the amount of cleaning needles backup to the required amount of 75 g, proceed as follows:

- Place the cleaning beaker (with cleaning needles, but without liquid) on a scales.
- A dry beaker and cleaning needles:
  - Add cleaning needles until the scales indicates a weight of between 144 g and 145 g.
- A wet beaker and cleaning needles:
  - Add cleaning needles until the scales indicates a weight of between 154 g and 155 g.

### 5.8. Rinsing the beaker cover

The cover of the beaker, especially the membrane (Fig. 9) and seals must be rinsed under clean water after each cleaning cycle for hygienic and functional reasons.

## 6. Cleaning and Maintenance

**The owner and user of the appliance are responsible for proper disinfection of the cleaning needles, cleaning beaker and lid, as well as the compliance with national hygiene regulations.**

- Clean the appliance using only a damp cloth.
- Wipe off the drip edge regularly, at least on a daily basis (Fig. 10).

**Never clean the appliance under running water or immerse in water.**

**Do not steam clean the appliance.**

**Do not use solvent-based, aggressive or scouring cleaners.**

### 6.1. Replacing cleaning needles

Replace the cleaning needles when the cleaning and polishing effect starts to subside.

To do this:

- Hold the retainer magnet against the underside of the cleaning beaker (Fig. 8).
- Carefully pour out the **activefluid cleaning liquid**.
- Rinse the cleaning beaker and the needles which are to be disposed of with water.
- Remove the retaining magnet and tip out the old needles.
- Rinse out the cleaning beaker.
- Fill with new cleaning needles (75 g, corresponding to one packaging unit).

Cleaned and disinfected cleaning needles can be disposed of with the regular recycleable waste.

## 6.2. Cleaning beaker.

Cleaning beakers are not heat-proof.

- ! **Inadequate cleaning of the beaker and lid can cause crystallisation of the activefluid cleaning liquid. This can result in the expansion of the beaker.**
- ! **Do not place cleaning beakers in a dish washer or sterilise in an autoclave.**
- ! **Cleaning beaker, lid and seal are subject to wear and tear. For this reason, check integrity of these parts before use.**

## 6.3. Changing a fuse

**Before replacing a fuse, disconnect the appliance from the mains.**

**Never use a fuse of larger impedance.**

The fuse is found next to the mains connection (N, Fig. 3).

- Disconnect from mains supply.
- Unlatch the fuse holder on both sides and remove.
- Remove faulty fuse and replace with a new one.
- Replace the fuse holder by sliding back into the socket until both sides lock into place.

## 7. Spare Parts

Refer to the spares list at the end of this manual for numbers of wearing and replacement parts.

## 8. Accessories

REF 150-020-00	<b>activefluid cleaning liquid</b> (1000 ml)
REF 907-944-30	Cleaning beaker
REF 098-001-00	Cleaning needles (75 g)

## 9. Troubleshooting

Error	Cause	Solution
<b>No display.</b>	Appliance not connected to mains.  Appliance not turned on.  No power from the socket. Faulty fuse.	Check correct fit of power cable in the appliance socket and wall socket. Turn on appliance at the mains switch (L, Fig. 3). Check power socket, for example, with another appliance. Replace the fuse (see section 6.3.). If the fuse blows repeatedly, have the appliance inspected by a service technician.
<b>Cannot change cleaning time.</b>	Appliance still running.	Interrupt the cleaning process with the Start/ Pause button, and then adjust cleaning time.
<b>Cannot save cleaning time.</b>	Cleaning time was entered during an interrupted cleaning process.	Stop the cleaning process with the Stop button, and then enter new cleaning time.
<b>Insufficient cleaning performance.</b>	Cleaning time too short. Active ingredients of the <b>activefluid cleaning liquid</b> spent. Insufficient number of, cleaning needles. Cleaning needles spent.	See chapt. 5.5. Prepare fresh <b>activefluid cleaning liquid</b> after each application. Fill up the cleaning needles (see section 5.7.2.). Replace the cleaning needles (see section 6.1.).
<b>Cleaning needles have corroded.</b>	Cleaning needles have been insufficiently rinsed. Cleaning needles have been stored incorrectly.  Cleaning needles in use for longer period of time.	Rinsing and storage see section 5.7.1. Removing the corrosion: Clean the needles with 100 ml <b>activefluid cleaning liquid</b> , without contents, for 5 min at 2000 min <sup>-1</sup> . Dispose of <b>activefluid cleaning liquid</b> .  The needles are so-called consumable goods. If it is not possible to regenerate the needles or the needles' performance deteriorates, see chapter 6.1.

The following information is designed to help you use the **activeblue high-performance cleaning device** safely in your laboratory or your dental office.

**Instruct the user according to the information in this manual on application, possible dangers, and use of the activeblue high-performance cleaning device.**

Ensure that this user information is available to operators.

## A. Application

### A.1 Correct use of the unit

The **activeblue high-performance cleaning device** is used in conjunction with **activefluid cleaning liquid** exclusively for professional mechanical cleaning of removable orthodontic appliances (e.g. braces) and splints. These must be fabricated using materials, which are approved for medical use (see orthodontic materials list at [www.dentaurum.de](http://www.dentaurum.de)). Particular exceptions are appliances made from CuNiZn (new silver) and silver solder connections.

The appliance is designed for non-sterile use only.

### A.2 Unapproved use

Use only Dentaurum original accessories and materials (such as **activefluid cleaning liquid**) with this appliance.

The use of other accessories is against intended use stipulations.

The cleaning of orthodontic appliances, fabricated using materials other than those approved for medical application, is also regarded as improper use.

### A.3 Ambient conditions

(according to DIN EN 61010-1)

The appliance should only be operated:

- indoors
- up to 2,000 metres above sea level
- at an ambient temperature of 5 °C - 40 °C [41 °F - 104 °F] \*)
- at a maximum relative humidity of 80 % at 31 °C [87.8 °F], linear reduction up to 50 % relative humidity at 40 °C [104 °F]\*)
- with mains electricity supply provided that the voltage fluctuation is within 10 % of the rated value
- with Pollution Degree 2
- with Overvoltage Category II

\*) The unit can be used at a temperature of 5 °C - 30 °C [41 °F - 86 °F] and at a humidity of up to 80 %. At temperatures of 31 °C - 40 °C [87.8 °F - 104 °F] the humidity must reduce proportionately to ensure that the unit can be operated (e.g. at 35 °C [95 °F] = 65 % humidity, at 40 °C [104 °F] = 50 % humidity). The unit should not be operated at temperatures above 40 °C [104 °F].

## B. Hazards and warnings

**Granted protection is not given any longer in case the unit is not operated according to the instructions for use on hand.**

**Only to be used indoors. The appliance is only intended for dry operation and should not be used or stored outdoors or in wet conditions.**

**The appliance should only be operated using a mains cable with a country-specific plug**

system. Any modification required should only be carried out by an electrician.

-  The appliance should only be operated if the data on the rating plate corresponds with the data of regional mains voltage.
-  The appliance should only be plugged into earthed sockets.
-  Regularly check connecting cables and hoses (e.g. mains cable) for damage (e.g. kinks, tears, porosity) or deterioration. Do not operate appliance with damaged mains wires, hoses or other defects.
-  Risk of injury! Risk of injury if the components used are not approved. Always use original Dentaurum components.
-  Cleaning needles contain a slight amount of nickel (9 % max),  
Dimensions: L = 6.5 - 7.5 mm,  $\varnothing$  0.39 - 0.41 mm).
-  Rinse the object under running water after cleaning and disinfect.
-  Check the object after cleaning and remove any adhering cleaning needles.
-  Safety advice: activefluid cleaning liquid is not classed as a harmful material according to GefStoffV / EC regulations.
-  First Aid measures: If activefluid cleaning liquid comes into contact with the eyes, rinse thoroughly immediately with plenty of water and consult a doctor, if necessary. After swallowing, rinse mouth out immediately and drink plenty of water. Do not induce vomiting. Consult a doctor immediately.
-  The liquids must not come into contact with the following surfaces: Natural stone and wooden surfaces.
-  Please do not let the cleaning beaker filled with cleaning fluid in the unit for a long period of time (e.g. overnight).
-  Damage or discolouration may occur should the activefluid cleaning liquid come into contact with other objects or laboratory furniture. This is valid particularly for surfaces out of PVC, linoleum, marble, natural stone, artificial stone, stainless steel and wooden surfaces. In case of contamination, immediate team with a copious amount of water.
-  Danger of scalding! Prolonged use of the appliance may cause the activefluid cleaning liquid to heat up. Always use of pliers to remove the object.
-  The appliance contains strong magnets! People with cardiac pacemakers have to keep 50 cm from the appliance. Do not place data storage mediums, credit cards or watches in the vicinity.

## C. Approved personnel

The activeblue high-performance cleaning device should be operated and maintained by trained personnel only.

## D. Prior to installation

Compare the data on the rating plate with the data of the regional mains voltage before installation.

The appliance should only be plugged into earthed sockets.

## E. Repair

Repairs should only be carried out by qualified personnel.

Repairs to the electrical equipment, which are not specified in these operating instructions, should only be carried out by an electrician.

## F. Disposal instructions

The appliance should be disposed of by a specialist firm. The specialist firm should be informed of any hazardous residue in the appliance.

**Crushing hazard! The unit contains strong magnets! Wear protective gloves for disassembly.**

**Risk of splitter! The magnets are brittle. Wear protective glasses for disassembly.**

### F.1 Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling. Within the European Union appliances with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

## G. Technical data

Rated voltage:.....	100 - 240 V / 50/60 Hz
Power consumption:.....	90 VA
Mains power fuse.....	T1,6AL / 250V
Cleaning RPM:.....	1200 / 1600 / 2000 min <sup>-1</sup>
Dimensions (W x H x D):.....	150 x 240 x 280 mm
Weight (without beaker) approx.:.....	approx. 3 kg
Noise level: .....	< 70 dB(A)

## H. Operating duration

Continuous operation for max 1 hour followed by 15 min interruption minimum.

## I. Disclaimer

Dentaurum GmbH & Co KG is not liable for claims for compensation or claims under guarantee if:

- **the object to be cleaned will be damaged by incorrect treatment.**
- **the product is used for purposes other than those stated in the instructions for use**
- **the product is modified in any way – apart from modifications described in the instructions for use**
- **the product has not been repaired by a specialist firm or original Dentaurum replacement parts have not been used**
- **there is continued use of the product despite obvious safety defects or damage**
- **the product has been subjected to mechanical knocks or has been dropped.**

## K. Guarantee

Used according to manufacturer's instructions, all parts of the **activeblue high-performance cleaning device** carry a 3-year guarantee.

The original sales invoice of the specialist depot is required for a claim under guarantee.

Parts subjected to normal wear and consumables (for example, the cleaning needles, Cleaning beaker, Lid, Seal) and tear are not covered by the guarantee.

The guarantee is rendered void in the case of incorrect use, non-adherence to the operating, cleaning, servicing or connection instructions, repairs carried out by the owner or repairs that are not carried out by a specialist firm, use of replacement parts from another manufacturer or unusual or unapproved uses not specified in the user guidelines.

Successful claims under guarantee do not extend the guarantee period.

**Full operational safety can only be guaranteed if Dentaurum system components (activeblue high-performance cleaning device, needles, beakers, activefluid cleaning liquid) are used exclusively.**

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