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Dear customer,

Thank you for choosing a quality product from Dentaaurum. Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaaurum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

Herbst TS is a construction with a telescope mechanism in three parts. This Herbst version is fitted with an inner telescope which prevents the hinge from sliding apart. The laser markings on the telescopic element indicating size and side guarantee clear allocation to the jaw.

3. Intended purpose

Correction of distocclusions.

4. Intended users

All products are intended for dental professionals only.

5. Indications

Distocclusion / class II (unilateral/bilateral), molar distalization, gap closure (aplasia), temporomandibular dysfunction, sleep apnea therapy, dentoalveolar compensation of occlusion (elastics substitution).

6. Contraindications

The SUS appliance made of stainless steel must not be used for patients with a nickel allergy. Warnings are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients for whom a fixed appliance can be placed as part of orthodontic treatment.

8. Fabrication of the Herbst TS appliance

Cast the impressions, which include the waxed out bands fitted on the first premolars and molars, with dental stone (e. g. Rapidur®). Mount the models in the correct position in an articulator with plaster using the construction bite recorded by the dentist. Bend and laser weld the half-round remaloy® wire (Ø 0.75 x 1.5 mm) or round remaloy® wire (Ø 1.0 mm) to the bands to form a lingual bow. A palatal bar can be placed in the upper jaw between the first molars to provide stabilization. Next laser weld the bases of the hinge joints to the bands. Ensure that the screws are at the correct angle to the axis between the upper and lower jaw. It is also important that the articulations between the upper and lower jaw operate without friction to ensure smooth functioning of the telescope mechanism. Friction can damage the appliance – especially the bands.

The telescopic tubes and telescopic rods of the Herbst TS retention hinge cannot be adjusted individually. The matching retention hinge is selected according to the construction bite by closing it and placing it on the base or by measuring the distance between the two bands beforehand.

Notes:

“Double bands” should be used to minimize the risk of the bands being damaged by shear forces. This involves welding a second band over the first band (e.g. using the spot welding unit Assistent 3000 or a laser). Generally the second band fits if it is 2 to 3 sizes larger than the first. Ready-welded “double bands” can be ordered from Dentaaurum. The order form is available on request.

Fitting intra-orally

The bands are fitted intra-orally using standard band cement. We recommend covering the bases of the articulations with wax to ensure that the cement does not penetrate into the thread. After curing of the cement the Herbst TS is unscrewed.

If necessary, the Herbst TS appliance can be activated using spacer rings, (please refer to our product range).

Since Herbst TS cannot be separated, place and crimp our slotted spacer rings on the lower section of the rod to continue mandibular advancement during the course of treatment.

9. Composition

The Herbst TS appliance is made of the following materials: DIN 1.4301, DIN 1.4305, DIN 1.4310 and DIN 1.4542. The material composition is included in the orthodontics materials list; please refer to the catalog or www.dentaaurum.com. The SSCP is available at https://ec.europa.eu/tools/eudamed and www.dentaaurum.com.

10. Safety information

The fit and the tightness of the screws should be regularly checked. If necessary, tighten the screws and secure them with a special screw securing adhesive, e. g. NOBIL GRIP. The loops of the Herbst TS must not be bent. It is not intended that the loops can be ground. The product should not be used if there is a known allergic reaction to one or more of the material components. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of. The Herbst TS telescope attachments are constructed in such a way that the stopper which holds the telescope rod in the middle telescope element can be overcome with higher force. It is therefore possible, in a medical emergency, to access the mouth and throat without need of special instruments in order to, for example, carry out emergency intubation.

11. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

12. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly. For reasons of biocompatibility, welding (e.g. with a laser welding unit) should be chosen over soldering when preparing the appliance.

13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

14. Product range

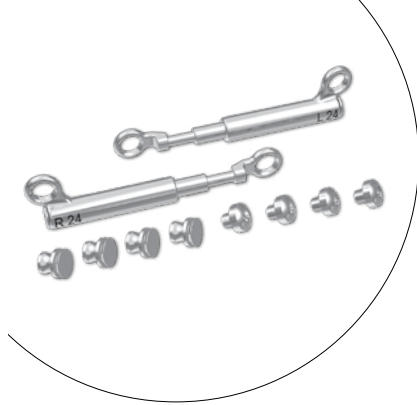
For the complete range of Herbst products, please see the orthodontic catalog or visit www.dentaaurum.com.

15. Quality information

Dentaaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as we, Dentaaurum, have no influence on the processing on site.

16. Explanation of symbols used on the label

Please refer to the label. The Instructions for use and additional information can be found at www.dentaaurum.com (Explanation of symbols REF 989-313-00).

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➤ For more information on our products, please visit www.dentaaurum.com

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