
Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso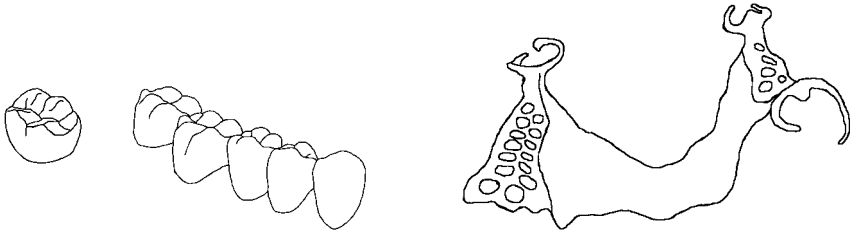

rema[®]
CAD/Vest 

Phosphatgebundene Einbettmasse für den 3D-Druck (CAD/Vest Verfahren)
Phosphate-bonded investment material for 3D printing (CAD/Vest process)
Revêtement à liant phosphate pour l'impression 3D (procédé CAD/Vest)
Revestimiento ligado por fosfato para la impresión 3D (técnica CAD/Vest)
Rivestimento a legante fosfatico per stampa 3D (sistema CAD/Vest)

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Dear customer,

Thank you for choosing a quality product from Dentaurum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit.

Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaurum.com, even though you may frequently use the same product.

Manufacturer

Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

Safety instructions

When using the speed preheating method, the furnace door must remain closed for at least 45 minutes after the casting rings have been placed inside (danger of burns from flames).

The investment material contains quartz. Do not inhale the dust! Risk of lung damage (silicosis / lung cancer)

Recommendation: wear protective face mask type FFP2 – EN 149:2001.

Cut open the sachet with a pair of scissors and avoid generating dust when emptying into the mixing bowl.

Rinse the empty sachet with water before crushing.

Use a damp cloth to remove dust at the workplace.

In order to avoid dust generation during deinvestment, soak the casting ring in water until it is completely moist.

When sandblasting, use a suction unit with a fine filter dust bag.

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Notes on processing

We recommend fitting a wax stick, 2 mm in diameter, to facilitate burn-out and degassing. This air duct connects the highest point of the printed object with the sprue former.

Roughen the top of the mould after the material has set.

Recommendation:

Remove the cylinder from the preheating furnace after 45 minutes with final temperature and blast compressed air into the opening of the sprue. Then place the casting ring back into the furnace for 15 minutes.

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EN

Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

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Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- Informationen zu Produkten und Serviceleistungen finden Sie unter www.dentaurum.com
- For more information on our products and services, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits et services sur www.dentaurum.com
- Descubra nuestros productos y servicios en www.dentaurum.com
- Informazioni su prodotti e servizi sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour |
Fecha de la información | Data dell'informazione: 04/21

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications |
Reservado el derecho de modificación | Con riserva di apportare modifiche