IN

Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

Federbolzenschrauben

Piston spring screws

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Tornillos con resorte

Viti a bullone

**D**  
**DENTAURUM**

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**Dear customer,**

Thank you for choosing a quality product from Dentaureum.

Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

**1. Manufacturer**

Dentaureum GmbH & Co. KG

Turnstr. 31 | 75228 Ispringen | Germany

**2. General device description**

The piston spring screw is a kind of threaded pin which holds a spring pin which extends beyond the tip. It is used for the movement of single teeth and is integrated into a removable orthodontic appliance with a serrated housing nut.

**3. Intended purpose**

Piston spring screws are used for single tooth movement as part of orthodontic treatment with a removable orthodontic appliance.

**4. Intended users**

All products are intended for dental professionals only.

**5. Indications**

Single tooth movement as part of orthodontic treatment with a removable orthodontic appliance.

## 6. Contraindications

Stainless steel expansion screws must not be used in patients with a nickel and/or chromium allergy. Warnings are found on the respective product labels. Also see 9. Composition.

## 7. Patient target group

All patients for whom a removable appliance with a piston spring screw can be used as part of orthodontic treatment.







## 8. Fabricating a removable appliance with a piston spring screw

### 8.1 Application of Orthocryl® using the salt-and-pepper technique

Insert the mounting screw into the serrated housing nut with approx. 0.5 mm projecting beyond the conical end. Position the mounting screw in the required position on the tooth and fix with sticky wax (ensure the model is dry for secure adhesion of the wax).

An alternative possibility: Drill a hole, 2.0 cm deep,  $\varnothing$  1.5 mm, in the tooth on the model and insert the mounting screw. The distance between the housing nut and the cervical area of the tooth should be approx. 0.5 mm.

Soak the model for 10 mins to 15 mins in lukewarm water in order to remove all air bubbles, then apply a layer of separating medium to the model. Now Orthocryl® can be applied to the model using the salt-and-pepper technique.

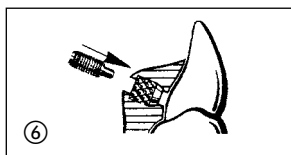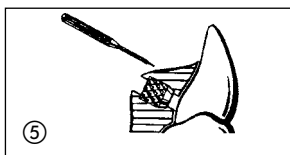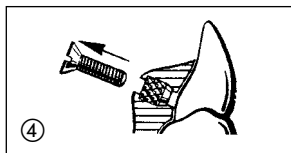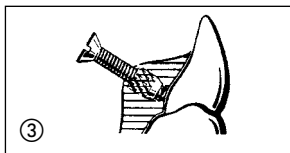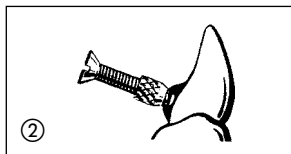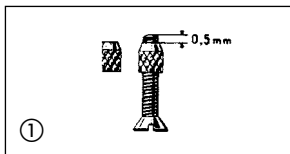## 8.2 Application of Orthocryl® LC

The mounting screw is fixed on the model in the same way as for the salt-and-pepper technique. Apply separating medium to the dry, not soaked model, then apply Orthocryl® LC in accordance with the Instructions for use and polymerize.

## 8.3 Finishing and use

Following polymerization, unscrew the mounting screw on the model, finish and polish the plate, then recut the thread with the reamer (REF 609-300-00).

Then insert the piston spring screw in the required size. Begin using the 4.0 mm screw, then exchange it for the 6.0 mm screw and once again for the 8.0 mm screw if the tooth in question needs to be moved so far.

**Note: The screwdriver (REF 609-400-00) is not a medical device and is not suitable for use in the mouth.**

## 9. Composition

The piston spring screws are made of the following materials: DIN 1.4305 and DIN 1.4310. The material composition is included in the orthodontics materials list; please refer to the catalog or [www.dentaurum.com](http://www.dentaurum.com).

## 10. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

## 11. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

## 12. Further information

The patient brochure "Oral hygiene during orthodontic treatment" (REF 989-328-20), available at [www.dentaurum.com](http://www.dentaurum.com), gives information on the correct cleaning and care of this and similar apparatuses.

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

### **13. Notes on disposal**

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

### **14. Product range**

For the complete product range of piston spring screws, please see our orthodontics catalog or [www.dentaurum.com](http://www.dentaurum.com).

### **15. Quality information**

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

### **16. Explanation of symbols used on the label**

Please refer to the label. These Instructions for use and additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

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