Touch Up

Photo: © Christian Ferrario

Das innovative Touch Up Konzept – Weniger Aufwand, mehr Ästhetik	2 – 7
The innovative Touch Up concept – Minimal effort, enhanced aesthetics	8 – 13
L'innovant concept « Touch Up » – Moins de travail et plus d'esthétique	14 – 19
El innovador Concept Touch Up – Menos esfuerzo, más estética	20 – 25

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Touch Up

Minimal effort, enhanced aesthetics – The innovative Touch Up concept

The completely user-friendly Touch Up concept produces excellent aesthetics and can also be used for a further range of outstanding applications. The Touch Up special ceramic materials complement the basic programme of the main line ceraMotion[®] ceramic system. The reduced firing temperatures provide a whole new range of possibilities of use. The Touch Up Opaque, Base Dentines, Dentines and Incisal materials complement the ceraMotion[®] ceramic system perfectly and are exactly suited.

enhanced Aesthetics

- individual degrees of glaze possible on a single piece of work
- smooth and homogeneous surfaces
- optimal marginal areas, occlusal surfaces and contact points resembling that of natural dentition

time saving due to less effort

- easy to process correction and glaze firing in one
- safe build-up on objects which have been previously inserted
- easy, quick repair due to reduced firing temperature

Dental Technician Hubert Dieker

Master Dental Technician Waldemar Fritzler

From users for users.

Touch Up

Creative firing/Additional firing:

a) Creative firing

- very natural marginal and incisal ridges
- customised degrees of glaze on a single restoration
- adjustment and glaze firing in one

Application of Touch Up material Dentin/Incisal, mixed with Modelling Liquid

Finished restoration

Note: Adjustments can be made with or without glaze material. When using glaze material, first cover the entire surface with glaze material, stain the restoration and then apply Touch Up material over the glaze material and staining.

	Start temp. (°C / °F)	Drying time (min)	Heat rate (°C / °F/min)	Vacuum start (°C / °F)	Vacuum end (°C / °F)	Final temp. (°C / °F)	Holding time	Long time cooling
Glaze and adjustment <u>with</u> glaze material	500 / 932	6	75 / 167	500 / 932	860 / 1550	860 / 1550	20 s	–
Glaze and adjustment <u>without</u> glaze material	500 / 932	6	75 / 167	500 / 932	860 / 1550	860 / 1550	20 s	–

b) Additional firing

- adding occlusal contact points at a later stage
- adding to the fitting surface

Application of Touch Up for missing occlusion

Adding to the fitting surface

Note: Adjustments can be made with or without glaze material. When using glaze material, first cover the entire surface with glaze material, stain the restoration and then apply Touch Up material over the glaze material and staining.

Touch Up

Repair

□ repair of a restoration worn by a patient

Prepare the framework

Finished restoration after the glaze firing

Note: Restorations that have been worn intraorally must be dried out in the preheat furnace. Clean the restoration, the surface must be roughened or sandblasted. Heat up the restoration in the preheating furnace from room temperature to 600 °C / 1112 °F at a rate of 5 °C / 41 °F/min. After a holding time of 2-4 hours, remove the object immediately from the furnace.

Apply the Opaque medium and bake according to the specified firing chart.

In case of repairs, please do mix 50 % Touch Up with 50 % Base Dentin, Dentin or Incisal.

	Start temp. (°C / °F)	Drying time (min)	Heat rate (°C / °F/min)	Vacuum start (°C / °F)	Vacuum end (°C / °F)	Final temp. (°C / °F)	Holding time	Long time cooling
Touch Up Opaque	500 / 932	8	75	500 / 932	860 / 1550	860 / 1550	20 s	–
Dentin firing during repair	500 / 932	6	75	500 / 932	860 / 1550	860 / 1550	20 s	–
Glaze firing during repair <u>without</u> glaze material	500 / 932	4	75	500 / 932	860 / 1550	860 / 1550	20 s	–
Glaze firing during repair <u>with</u> glaze material	500 / 932	6	75	500 / 932	860 / 1550	860 / 1550	20 s	–

Set / Product overview ceraMotion[®] Me Touch Up

ceraMotion [®] Me Touch Up Set	Quantity	REF
Touch Up Set	1 set	252-800-50
Contents:		
TU Paste Opaque	medium 3 g	252-250-03
TU Base Dentin	light 20 g	252-251-20
TU Base Dentin	medium 20 g	252-252-20
TU Base Dentin	dark 20 g	252-253-20
TU Dentin	light 20 g	252-255-20
TU Dentin	medium 20 g	252-256-20
TU Dentin	dark 20 g	252-257-20
TU Incisal	medium 20 g	252-260-20
TU Incisal	opal 20 g	252-261-20
TU Incisal	transpa 20 g	252-262-20
Shade Disc Touch Up	1 piece	254-103-00

ceraMotion [®] Me		
Touch Up Paste Opaque TU PO	Quantity	REF
Touch Up Paste Opaque TU PO medium	3 g	252-250-03
Touch Up Gingival		
Touch Up gingival TU G2	20 g	252-263-20
Touch Up gingival TU G4	20 g	252-264-20
Touch Up Base Dentin TU BD		
Touch Up Base Dentin TU BD light	20 g	252-251-20
Touch Up Base Dentin TU BD medium	20 g	252-252-20
Touch Up Base Dentin TU BD dark	20 g	252-253-20
Touch Up Dentin TU D		
Touch Up Dentin TU D light	20 g	252-255-20
Touch Up Dentin TU D medium	20 g	252-256-20
Touch Up Dentin TU D dark	20 g	252-257-20
Touch Up Incisal TU I		
Touch Up Incisal TU I medium	20 g	252-260-20
Touch Up Incisal TU I opal	20 g	252-261-20
Touch Up Incisal TU I transpa	20 g	252-262-20
Shade Disc Touch Up		
Shade Disc Touch Up	1 piece	254-103-00

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