Gebrauchsanweisung | Instructions for use |  
Mode d'emploi | Modo de empleo | Modalità d'uso

# Icr

CE 0123

## CONTEC Icr

Retaineradhäsiv

Retainer adhesive

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Adhesivo para retenedores

Collante per retainers

**D**  
DENTAURUM

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**Dear customer**

Thank you for choosing a quality product from Dentaureum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. The written instructions cannot cover all possible situations that can occur during use. In case of questions or ideas, please contact your local representative. As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the Internet at [www.dentaureum.com](http://www.dentaureum.com), even though you may frequently use the same product.

**1. Manufacturer**

SAREMCO Dental AG | Gewerbestrasse 4 | CH-9445 Rebstein | Switzerland

**2. Product description**

CONTEC Icr is a flowable, light-curing, ultra-fine hybrid composite. CONTEC Icr is free of TEGDMA (triethylene glycol dimethacrylate) and HEMA (hydroxyethyl methacrylate).

### 3. Composition

Barium glass, silanized, BisGMA (Bisphenol A glycerolate dimethacrylate), BisEMA (ethoxylated bisphenol A dimethacrylate), catalysts, inhibitors, additives.

### 4. Indication

Bonding of lingual retainers.

### 5. Contraindication

Known allergies to the above-mentioned methacrylates, camphor quinone and BHT (butylated hydroxytoluene).

### 6. Side effects

There are reported single incidents of contact allergies with products of similar composition.

**Note:** CONTEC Icr is free of TEGDMA (triethylene glycol dimethacrylate) and HEMA (hydroxyethyl methacrylate).

### 7. Interactions

Avoid base materials that may hinder polymerization because of their content. This applies to all phenolic compounds, e.g. ZOE (zinc oxide eugenol) or materials containing thymol.

## 8. Application, instructions for direct bonding of a canine-to-canine lingual retainer

### 8.1 Preparation of retainer

Adapt the retainer as required to the lingual canine-to-canine surfaces and set aside.

### 8.2 Etching of teeth

Clean the teeth with oil-free pumice powder or fluoride-free paste. Rinse with water and dry. To etch the teeth, use long cotton wool rolls in the lower area from canine to canine to isolate the vestibular and the buccal area. Apply phosphoric acid etching gel, e.g. CONTEC etching gel (REF 163-115-00), on the lingual surfaces of the teeth and leave for 30 seconds. If the dental enamel has been fluoridated, the etching gel should be left on a further 10 seconds. Rinse thoroughly with water, ensure the etching gel is removed completely and isolate again with cotton wool rolls. Dry thoroughly.

### 8.3 Priming of teeth

Place a small amount of CONTEC lc primer (REF 163-113-10) on a mixing pad. Apply a thin, even layer of primer on each tooth to be bonded. Light cure for approx. 20 seconds.

## 8.4 Bonding of the lingual retainer

Place dental floss in the interproximal area between the lateral incisors and the canines from lingual to buccal. Leave a loop facing lingually through which the lingual wire can be led. Bring the shaped retainer wire into the required position. Fix the wire in the correct position to the dental floss loops.

Pull the ends of the dental floss tight to pull the lingual wire towards the lingual surfaces of the lower incisors and canines. Spread CONTEC lcr on the teeth either from a mixing pad or directly from the syringe. The adhesive should flow over and around the wire. Apply a small drop of CONTEC lcr to each end of the wire. Only use as much material as is necessary. Light cure each drop 20 seconds long and remove the dental floss. Trim and polish if necessary.

## 9. Warnings/Precautions

- When using the light guide, follow the manufacturer's instructions regarding handling, correct use and information on eye protection.
- Should you be concerned that the incisors may rotate considerably, the lingual retainer can be fitted before the lower brackets are completely removed from the incisors. Otherwise, the lingual retainer can be fitted after debonding.
- The above-mentioned curing times are valid for polymerization lamps with a light intensity of at least 700 mW/cm<sup>2</sup>. The light-curing

time may vary depending on the source of light and instructions for use. The wavelength of the lamp must cover the range of 400–500 nm.

- Make sure the material is at room temperature before it is used.
- CONTEC Icr disposable tubes (REF 163-120-00) are intended for single use only and should be disposed of after each application. Close the lid of the syringe after use.
- Standard medical gloves do not offer protection against the sensitizing effect of methacrylate. Should the product come into contact with the glove, remove the glove and dispose of it. Wash your hands immediately with soap and water and put on new gloves. Contact your doctor if you experience an allergic reaction.

## 10. Storage

Protect light-curing products from strong light or intense heat. CONTEC Icr has been developed for use at room temperature (20–25 °C / 68–77 °F). Store at 4–28 °C (39–82 °F). Light sensitive. If the room temperature exceeds 28 °C (82 °F), store the product in a refrigerator. Do not deep-freeze!

## 11. Batch number and date of expiry

For identification purposes in case of queries, please quote the batch number. The expiry date can be found on the packaging. The product should not be used after the expiry date.

## **12. Precautions**

Use the correct lid to close each container after each use. For dental use only. Keep out of reach of children.

## **13. Emergency procedures**

Rinse thoroughly with water if the product comes into contact with the oral mucosa. Rinse thoroughly with water if the product comes into contact with the eyes. Consult an ophthalmologist.

## **14. Hygiene**

Use the application instrument for one patient only. Dispense the products at some distance from the dental chair to avoid contamination.

## **15. Additional information**

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

## 16. Warranty

Our warranty is limited to the quality of our products. If the product quality is flawed, only the value of the product will be reimbursed. We accept no legal liability for further damages as a result of non-compliance with the Instructions for use or as a result of improper handling or improper use of the product.

The user should check if the product is suitable for the case in hand before using the product. The user expressly accepts all risks involved in using the product and is solely responsible for any resulting damages.

## 17. Explanation of symbols on labels

 Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

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