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## EN

### Dear customer,

Thank you for choosing a quality product from Dentaurum. Please read these Instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at [www.dentaurum.com](http://www.dentaurum.com), even though you may frequently use the same product.

#### 1. Manufacturer

Dentaurum GmbH & Co. KG · Turnstr. 31 · 75228 Ispringen · Germany

#### 2. Product description

The facebow consists of the inner bow positioned intra-orally and the outer bow positioned extra-orally. Both bows are laser-welded together in the middle.

#### 3. Intended purpose

As part of the headgear, the facebow serves as extra-oral anchorage and/or dental or skeletal correction of vertical and sagittal deviations.

#### 4. Intended users

All products are intended for dental professionals only.

#### 5. Indications

Used as part of a headgear, the facebow can aid distalization in the upper posterior region, secure anchorage and support skeletal inhibition in the maxilla.

#### 6. Contraindications

The Standard facebow may not be used for patients with a nickel allergy. The nickel-free Noninium® Facebow medium is a possible alternative. Warnings are found on the respective product labels. See also 10. Composition.

#### 7. Patient target group

All patients for whom a fixed appliance can be used as part of orthodontic treatment.

#### 8. Area of application

The facebow serves as extra-oral anchorage and/or dental or skeletal correction of vertical and sagittal deviations.

#### 9. Safety information for the orthodontist

You have prescribed your patient the facebow and the headgear for orthodontic treatment.

It can only be worn by the patient after instruction from you.

If used improperly or removed carelessly and without loosening the flexible bands of the headgear, the facebow may snap back suddenly and cause serious harm.

For safety reasons and **for the safety of your patients, only headgear with safety modules may be used!**

The headgear and the facebow may only be fitted and handed over by the orthodontist or dentist.

#### 10. Composition

The composition is included in the materials list, refer to the catalog or [www.dentaurum.com](http://www.dentaurum.com).

#### 11. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components.

#### 12. Information for single use products

The product is intended for single use only. Reconditioning of a product that has already been used (recycling) and its reuse on a patient is not permitted.

#### 13. Information on disinfection

Disinfection before initial use is not necessary.

#### 14. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and – if available – the details outlined in the safety data sheets.

#### 15. Product range

Facebows are available in the sizes mini, medium and maxi. Please refer to our Orthodontics Catalog for further details.

#### 16. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

#### 17. Quality information

Dentaurum ensures faultless quality of its products. The content of these Instructions for use is based upon our own experiences. The dental professional is solely responsible for the processing of the products. Liability for failures cannot be taken, as Dentaurum has no influence on the processing on site.

#### 18. Explanation of symbols used on the label

 Please refer to the label. Additional information can be found at [www.dentaurum.com](http://www.dentaurum.com) (Explanation of symbols REF 989-313-00).

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### Safety information for the patient / accident prevention

1. Follow the instructions of your orthodontist/dentist when using the facebow and headgear. It is important to follow the orthodontist's/dentist's instructions when fitting the headgear and facebow.
2. Always remove the flexible bands from the headgear before fitting or removing the facebow.
3. Do not let anyone pull on your headgear or facebow – you may be injured!
4. Never pull the facebow in an upward movement over your face; always remove it by pulling it horizontally from your mouth.
5. Never wear the facebow and the headgear during sport.
6. Contact your orthodontist/dentist if you have questions on using the facebow and headgear.
7. If used improperly, the facebow may cause injury.
8. Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

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