Estimado cliente:

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Dear customer,

Thank you for choosing a quality product from Dentaurum. Please read these instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaurum GmbH & Co. KG
Turnstr. 31 · 75228 Ispringen · Germany

2. General device description

The Neo BJA is a three-dimensional Bertoni expansion screw with guide ridges on the forward thrust double plate acc. to F.M. Sander. It is fitted into a removable orthodontic appliance and can correct skeletal malocclusions class II/1 and class II/2 by means of the sagittal expansion element.

3. Intended purpose

The Neo BJA (removable orthodontic appliance) is used to correct the maxillomandibular relationship in patients with a skeletal malocclusion class II/1 or class II/2.

4. Intended users

All products are intended for dental professionals only.

5. Indications

Skeletal malocclusions class II/1 or class II/2.

6. Contraindications

Stainless steel expansion screws must not be used in patients with a nickel and / or chromium allergy. Warnings to this purpose are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients who are still growing and for whom a removable appliance in the form of a bite jumping appliance can be used as part of orthodontic treatment.

8. Fabrication of the elastic bite jumping appliance

The first images show the Neo BJA (Fig. 1), the mounting template (Fig. 2) and the guide template (Fig. 3).

Mount the models with the construction bite in the fixator (Fig. 4). Attach elements for retention and movement and, if required, further expansion screws used (except for the Neo BJA) onto the models.

Use the mounting template (Fig. 2) to adjust the angle of the guide ridges or the Neo BJA (55°, 60°, 65°). Slide the mounting template into the groove of the expansion screw holder (Fig. 1).

Place the mounting template with the mounted Neo BJA parallel to the occlusal plane onto the maxillary model and align it with the midline of the maxilla. Shorten the cone of the expansion screw holder (of the Neo BJA) to suit the situation and fix it to the model with sticky wax or thermowax (REF 120-170-00) (Fig. 5a). Alternatively, the cone can be countersunk into a hole in the model (Fig. 5b).

Wax the screw for the mandible, aligned with the midline of the mandible, as closely as possible to the alveolar process or fix the screw by milling a hole in the model. Then shorten the expansion screw holder close to the screw (Figs. 6a + 6b). Remove the mounting template, connect the models in the fixator and verify the position of the guide ridges in relation to the mandibular screw. The guide ridges should each be at a distance of approx. 2 mm to the mandibular expansion screw (Figs. 7 + 8).

Fabricate the maxillary plate using the salt-and-pepper technique and polymerize this in a pressure vessel. It is an advantage to add a thin layer of thermo wax to the area of transition from the screw body to the expansion screw holder if the acrylic is moist during processing. It is easier to remove the expansion screw holder if it is not covered with acrylic from the top (Fig. 9).

Place the guide template with the flat side facing the acrylic onto the guide ridges of the Neo BJA (Figs. 3 + 10). Cover the acrylic of the maxillary plate in the anterior tooth region with a thin layer of wax to avoid the plates from joining together during polymerization (Fig. 10).

Fabricate the mandibular plate using the salt-and-pepper technique. Take care to apply a slightly thicker layer of acrylic at the front, around the screw. Press the upper fixator arm with the maxillary plate and the guiding template carefully into the acrylic of the mandible. Trim the surplus acrylic and polymerize the mandibular plate in the pressure vessel (Fig. 11).

Make the saw cuts for the sagittal and transversal activation of the expansion screws. Finish and polish the maxillary and mandibular plates.

Check the position of the guide ridges in relation to the mandibular screw. Remove the fixator. The Neo BJA is finished (Fig. 12).

9. Composition

The material composition is included in the orthodontics materials list; please refer to the orthodontics catalog or www.dentaurum.com.

10. Safety information

The product should not be used if there is a known allergic reaction to one or more of the material components.

It is the dental professional's responsibility to decide on combining different products with regard to the compatibility of chemical, physical and mechanical properties. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

11. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

12. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

13. Notes on disposal

When disposing of the product, please adhere to the national regulations that are valid for your region and - if available - the details outlined in the safety data sheets.

14. Product range

For the complete range of products for the Neo BJA acc. to F.M. Sander, please see our catalog or visit www.dentaurum.com.

15. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The dental professional is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaurum has no influence on the processing on site.

16. Explanation of symbols used on the label

Please refer to the label. These Instructions for use and additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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