STEP-BY-STEP

Orthocryl[®] LC

Easy and fast!
Facile et rapide !
¡Sencillo y rápido!
Semplice e veloce!

D
DENTAURUM

EN

Working with Orthocryl® LC is extremely easy:
We will show you step by step how to fabricate
an orthodontic appliance.

 The Orthocryl® LC program

Step-by-step instructions for the fabrication of orthodontic expansion plates

-  1. Preparing the model
-  2. Application of the separating medium
-  3. Preparing the material
-  4. Application of the material
-  5. Using different colors
-  6. Using Orthocryl® Disco glitter
-  7. Using decals
-  8. Polymerization
-  9. Removing the inhibition layer
-  10. Finishing the appliance

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Orthocryl[®] LC

EN 1. Preparing the model

In general, the following applies: Apply light-curing orthodontic acrylic Orthocryl® LC on a dry, not soaked, coated plaster model.

Mark the position of the expansion screws on the model and drill a hole for the screw holder with a conical bur (e.g. REF 123-598-00).

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Orthocryl[®] LC

EN 1. Preparing the model

Fill the hole on the model with heated wax (e.g. REF 119-150-00) and insert the expansion screw with the lower, basal part of the holder.

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Orthocryl[®] LC

EN 1. Preparing the model

Remove expansion screw after wax has hardened.

Fix bent wire elements to the model with sticky wax (REF 122-300-00).

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Orthocryl[®] LC

EN 2.1. Application of the separating medium (162-800-00) on plaster models

Apply separating medium from the dosing bottle (REF 162-100-00) directly on the dry model and spread evenly using a paintbrush.

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Orthocryl[®] LC

EN 2.1. Application of the separating medium (162-800-00) on plaster models

Stand the model onto its back (tuber-plane) so that excess fluid can drain away. After approx. 30 seconds, carefully blow away separating medium residues or droplets with compressed air and, if necessary, remove residues from screws and wire retentions with a dry paintbrush.

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EN 2.2. Application of the 3D separating medium (162-801-00) on plastic models

Apply the 3D separating medium evenly with a brush over the entire model before the expansion screw(s) and wire elements are finally fixed in place and allow the separating medium to dry thoroughly. Apply a second layer.

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Orthocryl[®] LC

EN 3. Preparing the material

Remove small and large blue protective cap on Orthocryl® LC cartridge by turning it counterclockwise.

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Orthocryl[®] LC

EN 3. Preparing the material

Rotate a replaceable cartridge tip onto the top of the cartridge as far as it will go and shorten it to the correct length.

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Orthocryl[®] LC

EN 3. Preparing the material

The button on the upper side of the injector frees the forward and back feed of the injector plunger. With the button pressed, bring the plunger of the injector into initial position.

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Orthocryl® LC

EN 3. Preparing the material

Stick the cartridge onto the injector and lock it with a 90° turn. With the button on the upper side pressed, push forward the plunger of the injector up to the stop.

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Orthocryl[®] LC

EN 4. Application of the material

Apply escaping material carefully on the model, wires and screws. By pushing the trigger on the injector, material escapes from the cartridge.

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Orthocryl® LC

EN 4. Application of the material

Surround the removed expansion screw completely with Orthocryl® LC. Embed the expansion screw completely in the acrylic.

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Orthocryl[®] LC

EN 4. Application of the material

Afterwards, insert the expansion screw back into the drilled hole with the wax coating. Due to the wax, the expansion screw will regain the same position as determined before.

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Orthocryl[®] LC

EN 4. Application of the material

With wire retentions, apply Orthocryl® LC to the space between wire and plaster model first and then embed the complete wire.

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Orthocryl[®] LC

EN 4. Application of the material

Apply Orthocryl® LC until the desired form and thickness of the appliance (maxilla approx. 3 mm / mandible approx. 4 - 5 mm) is achieved.

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Orthocryl[®] LC

EN 4. Application of the material

Should this process require more than a single step, slightly dip the tip of the cartridge into the already applied material, when starting the following step. This prevents air inclusions between the layers.

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Orthocryl[®] LC

EN 5. Using different colors

For interim storage, close the color cartridges that are not in use with the blue caps on the top and bottom. Simply insert a new color in the injector.

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Orthocryl[®] LC

EN 5. Using different colors

Use a new cartridge tip for each cartridge.

In order to guarantee secure hardening of the color layers, we recommend applying a layer of clear Orthocryl® LC first.

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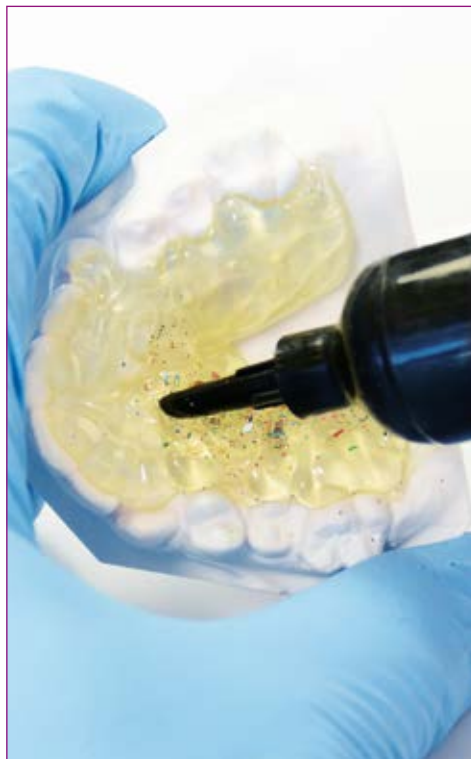
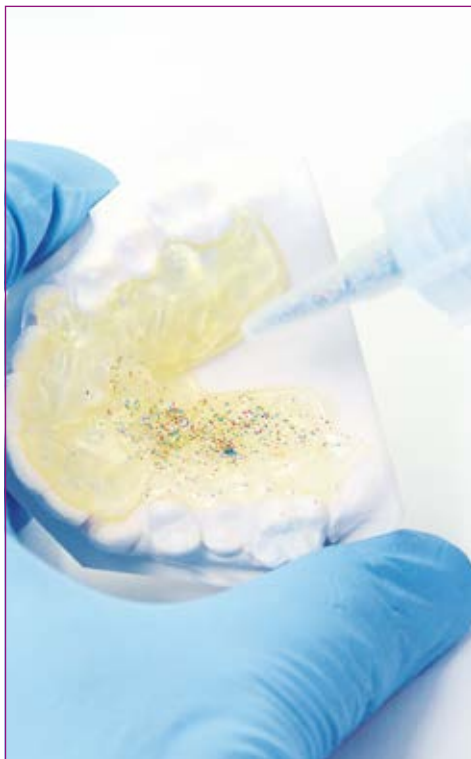
Orthocryl[®] LC

EN 6. Using Orthocryl® Disco glitter

Spray a bit of Orthocryl® Disco glitter on the first layer of Orthocryl® LC directly from the spray bottle.

Then, cover it with another layer of Orthocryl® LC.

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Orthocryl[®] LC

EN 7. Using decals

Select decal from the extensive program and if necessary cut to the correct size.

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Orthocryl[®] LC

EN 7. Using decals

Place the cut decal on the first Orthocryl® LC layer, at about the height of the wire retentions.

Then, cover it with another layer of Orthocryl® LC.

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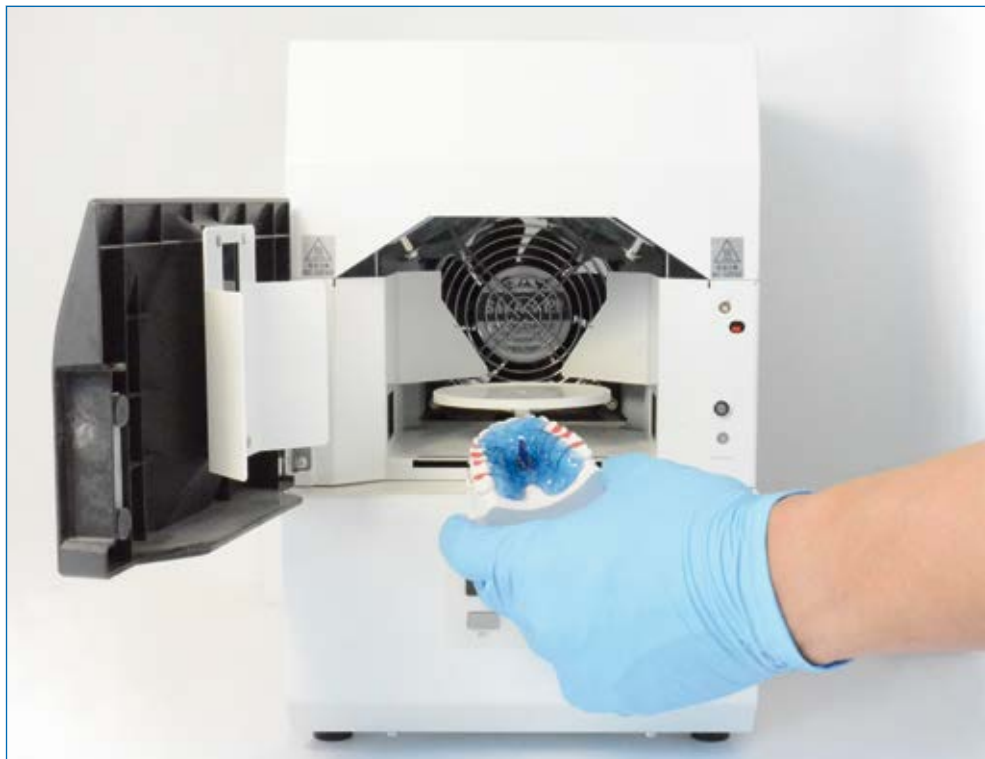
Orthocryl[®] LC

EN 8. Polymerization

For polymerization, use a light-curing unit with a 400 – 550 nm wave length and a 4 x 150 watt (600 watt) light output. We recommend the Solidilite V unit by SHOFU Dental GmbH. Observe the instructions for use of the respective unit.

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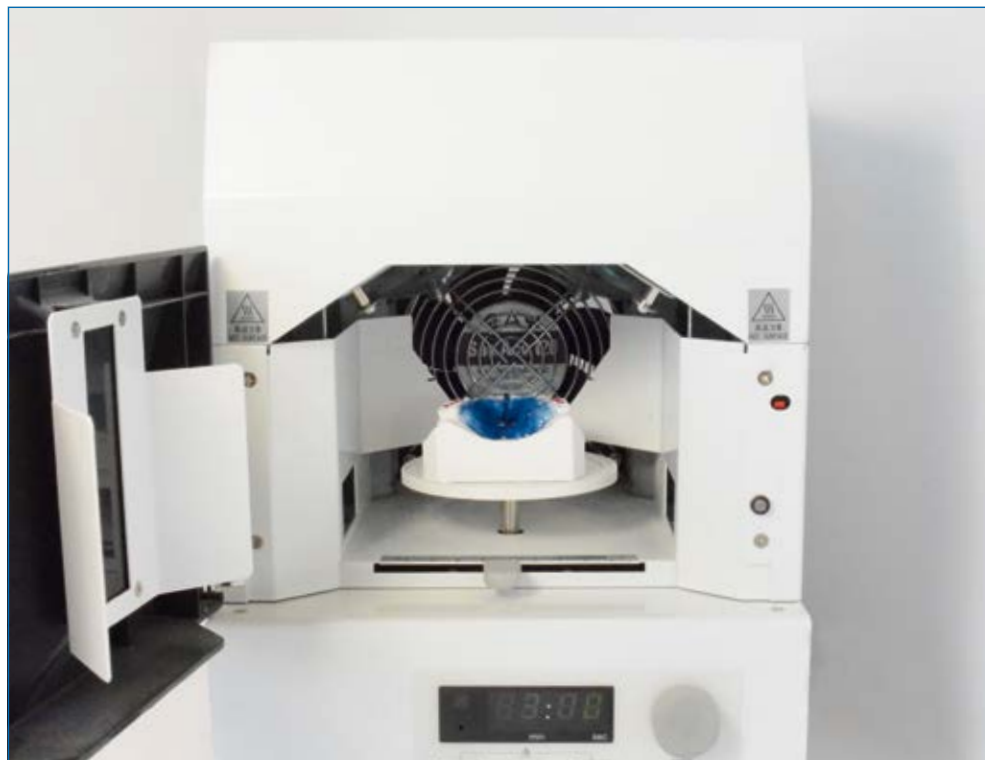
Orthocryl[®] LC

EN 8. Polymerization

The rotary plate in the curing chamber is height-adjustable. During polymerization, the rotary plate should be at the highest position possible.

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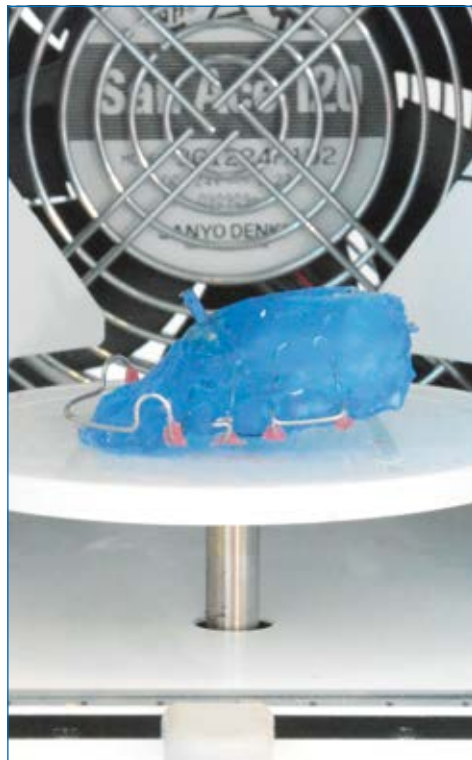
Orthocryl[®] LC

EN 8. Polymerization

Afterwards, carefully remove the appliance from the model.

Cure the appliance for another **3 minutes** with the basal side up.

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Orthocryl[®] LC

EN 9. Removing the inhibition layer

During polymerization, an air-inhibition layer (smear layer) is formed. Completely remove this layer. Apply Orthocryl® LC cleaning liquid (REF 160-430-00) on a piece of fabric and remove the smear layer.

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Orthocryl[®] LC

EN 10. Finishing the appliance

Finish Orthocryl® LC appliances using hard metal burs with transversal toothing or burs for soft acrylics. Cross-toothed burs are only suited to a limited extent. Prepolish appliance with silicone polishers or sand paper.

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Orthocryl[®] LC

EN 10. Finishing the appliance

Prepolish appliance with pumice powder and polishing brushes.

Achieve a high polish finish with the acrylic polishing material „Edelweiß“ (REF 190-100-00 or REF 190-200-00) and a cloth polishing brush.

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L'apparecchio finito.

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- ➔ For more information on our products and services, please visit www.dentaurum.com
- ➔ Vous trouverez toutes les informations sur nos produits et services sur www.dentaurum.com
- ➔ Descubra nuestros productos y servicios en www.dentaurum.com
- ➔ Ulteriori informazioni su prodotti e servizi sono disponibili nel sito www.dentaurum.com

Date of information / Mise à jour / Fecha de la información / Data dell'informazione: **2024-01**

Subject to modifications / Sous réserve de modifications / Reservado el derecho de modificación / Soggetto a variazione

D
DENTAURUM

Turnstr. 31 | 75228 Ispringen | Germany | Phone +49 72 31/803-0 | Fax +49 72 31/803-295
www.dentaurum.com | info@dentaurum.com