
Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

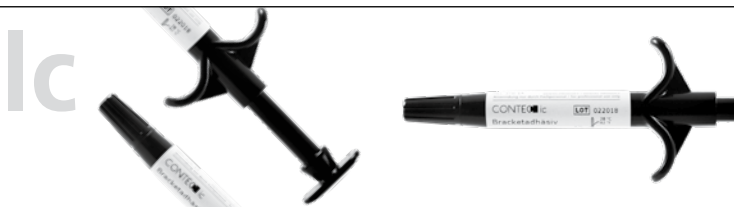CE 0123

CONTEC  Ic

Bracketadhäsiv

Bracket adhesive

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Adhesivo para brackets

Collante per brackets

DENTAURUM

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Dear customer

Thank you for choosing a quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. The written instructions cannot cover all possible situations that can occur during use. In case of questions or ideas, please contact your local representative. As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the Internet at www.dentaaurum.com, even though you may frequently use the same product.

1. Manufacturer

SAREMCO Dental AG | Gewerbestr. 4 | CH-9445 Rebstein | Switzerland

2. Product description

CONTEC Ic is a flowable, light-curing, ultra-fine hybrid composite. Both CONTEC Ic and CONTEC Ic primer are free of TEGDMA (triethylene glycol dimethacrylate), HEMA (hydroxyethyl methacrylate) and BisGMA (bisphenylglycidyl dimethacrylate).

3. Composition

CONTEC Ic:

barium glass, silanized, BisEMA (ethoxylated bisphenol A dimethacrylate), fumed silica, catalysts, inhibitors, additives

CONTEC Ic primer:

BisEMA (ethoxylated bisphenol A dimethacrylate), barium glass, silanized, catalysts, inhibitors, additives

4. Indication

Direct bonding of orthodontic brackets in metal, ceramic and plastic.

5. Contraindication

Known allergies to the above-mentioned methacrylates, camphor quinone and BHT (butylated hydroxytoluene).

6. Side effects

There are reported single incidents of contact allergies with products of similar composition.

Note: CONTEC Ic is free of TEGDMA, HEMA and BisGMA.

7. Interactions

Avoid materials that may hinder polymerization because of their content. This applies to all phenolic compounds, e.g. ZOE (zinc oxide eugenol) or materials containing thymol.

8. Application

8.1 Etching of teeth

Clean the teeth with oil-free pumice powder or fluoride-free paste. Rinse with water and dry. To etch the teeth, use long cotton wool rolls in the

lower area from canine to canine to isolate at least the vestibular area. Apply phosphoric acid etching gel, e.g. CONTEC etching gel (REF 163-115-00), on the surfaces of the teeth and leave for 30 seconds. If the dental enamel has been fluoridated, the etching gel should be left on a further 10 seconds. Rinse thoroughly with water, ensure the etching gel is removed completely and isolate again with cotton wool rolls. Dry thoroughly.

8.2 Priming of teeth

Place a small amount of CONTEC Ic primer on a mixing pad. Apply a thin, even layer of primer on each tooth to be bonded. Light cure for approx. 20 seconds.

8.3 Bonding the brackets

Apply a small amount of CONTEC Ic to the base of the brackets using the syringe. Use the adhesive sparingly. When this step is completed, wipe the tip of the syringe and close the lid. When bonding plastic brackets, use a primer especially for plastic brackets, e.g. CONTEC plastic primer (REF 163-100-10).

8.4 Application and light curing

Place the brackets gently on the surface of the tooth immediately after the adhesive has been applied. Then move the brackets into the final position and press firmly onto the tooth.

Note: If there is a delay in the final positioning, cover the patient's mouth with a dark mask or something similar to prevent the adhesive from curing due to the natural light.

Carefully remove surplus adhesive from the bracket base without dislodging the bracket.

Light cure each bracket for a total of 20 seconds (metal brackets: mesial and distal; ceramic and plastic brackets: through the bracket). Hold the light guide 2–3 mm away from the tooth; hold it above the interproximal contact point for metal brackets, and perpendicularly for ceramic and plastic brackets.

Tip for faster curing with metal brackets: Hold the light guide in the interproximal area between two brackets. Shine the light on both sides of the bracket to cure the adhesive completely.

Archwires can be fitted on the brackets immediately following curing on the last bracket.

9. Warnings/Precautions

- When using the light guide, follow the manufacturer's instructions regarding handling, correct use and information on eye protection.
- The above-mentioned curing times are valid for polymerization lamps with a light intensity of at least 700 mW/cm². The light-curing time may vary depending on the source of light and instructions

for use. The wavelength of the lamp must cover the range of 400–500 nm.

- Make sure the material is at room temperature before it is used.
- Close the lid of the syringe after use.
- Standard medical gloves do not offer protection against the sensitizing effect of methacrylate. Should the product come into contact with the glove, remove the glove and dispose of it. Wash your hands immediately with soap and water and put on new gloves. Contact your doctor if you experience an allergic reaction.

10. Storage

Protect light-curing products from strong light or intense heat. CONTEC Ic has been developed for use at room temperature (20–25 °C / 68–77 °F). Store at 4–28 °C (39–82 °F). Light sensitive. If the room temperature exceeds 28 °C (82 °F), store the product in a refrigerator. Do not deep-freeze!

11. Batch number and date of expiry

For identification purposes in case of queries, please quote the batch number. The expiry date can be found on the packaging. The product should not be used after the expiry date.

12. Precautions

Use the correct lid to close each container after each use. For dental use only. Keep out of reach of children.

13. Emergency procedures

Rinse thoroughly with water if the product comes into contact with the oral mucosa. Rinse thoroughly with water if the product comes into contact with the eyes. Consult an ophthalmologist.

14. Hygiene

Use the application instrument for one patient only. Dispense the products at some distance from the dental chair to avoid contamination.

15. Additional information

Should the user and/or the patient gain knowledge of serious incidents arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed.

16. Warranty

Our warranty is limited to the quality of our products. If the product quality is flawed, only the value of the product will be reimbursed. We accept no legal liability for further damages as a result of non-compliance with the Instructions for use or as a result of improper handling or improper use of the product.

The user should check if the product is suitable for the case in hand before using the product. The user expressly accepts all risks involved in using the product and is solely responsible for any resulting damages.

17. Explanation of symbols on labels

 Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00).

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