Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

CE 0123

CONTEC go

Bracketadhäsiv

Bracket adhesive

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Adhesivo de brackets

Collante per brackets

D
DENTAURUM

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Dear customer,

Thank you for choosing a quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaaurum.com, even though you may frequently use the same product.

1. Manufacturer

SAREMCO Dental AG | Gewerbestr. 4 | CH-9445 Rebstein | Switzerland

2. Product description

CONTEC go adhesive is an ultra-fine hybrid composite that is thixotropic and flowable. Both the CONTEC go adhesive and the CONTEC go Primer contain no TEGDMA and no HEMA.

3. Composition

CONTEC go adhesive:

barium glass, silanized, BisEMA, fumed silica, bead polymer, catalysts, inhibitors, additives

CONTEC go primer:

BisEMA, BisGMA, catalysts, inhibitors, additives

4. Indication

Direct bonding of orthodontic brackets in metal, ceramic and plastic.

5. Contraindication

Known allergies to the above-mentioned methacrylates, benzoyl peroxide, tert. amine and BHT.

6. Adverse reactions

There have been reported single incidents of contact allergies with products of similar composition.

Note: CONTEC go is free of TEGDMA and HEMA.

7. Interactions

Avoid materials that may hinder polymerization because of their content. This applies to all phenolic compounds, e.g. preparations containing ZnO-eugenol or thymol.

8. Application

8.1 Etching of teeth

Clean the teeth with oil-free pumice powder or paste. Rinse with water. Dry the teeth carefully with air, thereby creating a dry area. Dry the tooth with cotton wool rolls.

To etch the teeth, use long cotton wool rolls in the lower area from canine to canine to isolate at least the buccal area.

Apply phosphoric acid etching gel, e.g. CONTEC etching gel (REF 163-115-00), on the surfaces of the teeth. Observe the manufacturer's instructions for the etching gel or etching liquid. Etch for 30 seconds. Rinse carefully with water and suction off. The patient should not rinse his/her mouth. Carefully dry the etched enamel surfaces of the tooth with dry, oil-free air. The surface should look frost-white. If not, repeat the etching process.

8.2 Priming of teeth

Place a small amount of CONTEC go primer on a mixing pad. Apply a thin, even layer of primer on each tooth to be bonded and on the base of each bracket to be bonded.

8.3 Bonding the brackets

Apply a small amount of the CONTEC go adhesive to the primed bracket base. Use the adhesive sparingly. When finished, wipe a small amount of adhesive from the tip of the syringe and close with the cap.

Place the bracket on the tooth: move the bracket to the correct position within 20 seconds. Press the bracket onto the tooth surface to ensure it adheres correctly. Remove excess adhesive, making sure the bracket does not move! Hold in the final position for 5 – 10 seconds. Do not reposition. Allow the adhesive to cure for at least 5 minutes before ligating in the archwire.

9. Warnings/Precautions

- Make sure the material is at room temperature before it is used.
- Close the lid of the syringe after use.
- Standard medical gloves do not offer protection against the sensitizing effect of methacrylate. Should the product come into contact with the glove, remove the glove and dispose of it. Wash your hands immediately with soap and water and put on a new glove. Contact your doctor if you experience an allergic reaction.

10. Storage

The CONTEC go adhesive and CONTEC go primer have been developed for use at room temperature (20 °C – 25 °C / 68 °F – 77 °F). Store at 4 °C – 25 °C (39 – 77 °F). If the room temperature exceeds 25 °C (77 °F), store the product in a refrigerator. Do not deep-freeze!

11. Batch number and date of expiry

For identification purposes in case of queries, please quote the batch number. The product should not be used after the expiry date.

12. Precautions

Use the correct lid to close each container after each use. For dental use only. Keep out of reach of children.

13. Emergency procedures

Rinse thoroughly with water if the product comes into contact with the oral mucosa. Rinse thoroughly with water if the product comes into contact with the eyes. Consult an ophthalmologist.

14. Hygiene

Use the application instruments for one patient only. Dispense the products at some distance from the dental chair to avoid contamination.

15. Further information

Should the user and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident is informed accordingly.

16. Warranty

Our warranty is limited to the quality of our products. If the product quality is flawed, only the value of the product will be reimbursed. We accept no legal liability for further damages as a result of non-compliance with the Instructions for use or as a result of improper handling or improper use of the product.

The user should check if the product is suitable for the case in hand before using the product. The user expressly accepts all risks involved in using the product and is solely responsible for any resulting damages.

17. Explanation of symbols on labels

 Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00)

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