Gebrauchsanweisung | Instructions for use |
Mode d'emploi | Modo de empleo | Modalità d'uso

Wärmeplatte für Polyclav® Drucktopf
Heating plate for Polyclav® pressure vessel
Plaque chauffante pour autoclave de polymérisation Polyclav®
Placa calefactora para recipiente a presión Polyclav®
Piastra elettrica per pentola a pressione Polyclav®

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Druckdatum: 14.10.15

Dear customer

Thank you for choosing a quality product from Dentaureum.

It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application. In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current instructions for use supplied with the product and stored in the internet at www.dentaureum.com, even though you frequently use the same product.

1. Manufacturer

Dentaureum GmbH & Co. KG | Turnstraße 31 | 75228 Ispringen | Germany

2. Safety information

Read through the instructions for installing and operating the unit carefully before switching it on.

- The Heating Plate should only be used for heating the Polyclav® polymerization pressure vessel.
- Do not place inflammable substances such as cardboard or paper on the Heating Plate.
- Never use the Heating Plate without the Polyclav® pressure vessel.
- If repair is required: remove the electric supply plug before opening the unit.

3. General notes and disposal

When a Dentaureum product is finally no longer in use, the disposal regulations of that particular country apply. Dentaureum or the dental trade is available to answer questions regarding the correct disposal of any specific product.

Note:

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaureum or the dental trade.

4. Application and Description

The thermostatically controlled Heating Plate for the Polyclav® retains the constant temperature that is necessary for the polymerization process. The temperature of the water in the Polyclav® pressure vessel is kept at approx. 43 °C / 109 °F.

5. Technical Data

Voltage	230 V	50/60 Hz	REF 070-900-00
	115 V	50/60 Hz	REF 070-901-00
Power	125 W		
Dimensions	240 x 240 x 75 mm / 9 1/2" x 9 1/2" x 3"		

6. Operation of the Unit

- The heating plate is connected to the power supply via a grounded plug.

 **Important: The power supply voltage must correspond to the voltage stated on the plate of the unit.**

- The unit is switched on using the power supply switch.
- During the heating phase, the red light glows.
- Switch the unit off after use.

 **Important: Never use the unit without the Polyclav® pressure vessel.**

7. Care and Maintenance

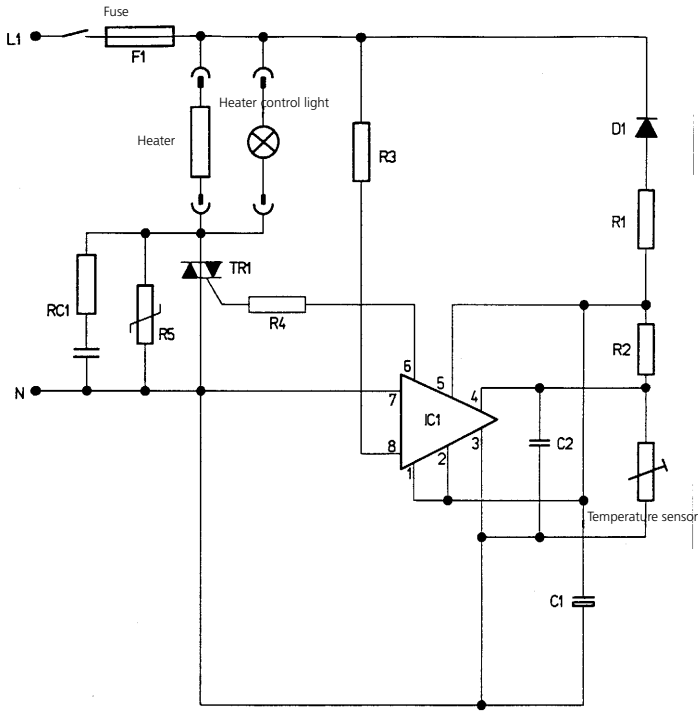
The heating plate requires very little care or maintenance. Use a damp cloth to clean the unit.

 **Note: Never use abrasive or corrosive cleaning agents to clean the unit.**

8. Spare Parts

	Unit, 230 V	Unit, 115 V
	070-900-00	070-901-00
Description	REF	REF
Rocker switch	907-143-10	907-143-10
Control module	978-120-10	978-120-20 (circuit board inside unit)
Temperature sensor	907-070-10	907-070-10

9. Circuit Diagram

10. Quality information

Dentaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

11. EEC-Declaration of Conformity

DENTAURUM GmbH & Co. KG
Turnstr. 31
75228 Ispringen

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EEC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

Description of unit:	Electric Heating Ring H1 for Polyclav	
REF:	070-900-00 / 070-901-00	
Start with unit No.:	13-4032 / 24-4111	
EEC guidelines:	89/336/EEC	Electromagnetic compatibility
	92/031/EEC	Modified Guideline
	73/023/EEC	Electrical equipment used within certain voltage limits
	93/068/EEC	Modifications of the above guidelines
Applied unified standards:	EN 60335	
	EN 55011/03.91	
	EN 50082-1/01.92	
	EN 61326:1997+A1:1998	

Date and manufacturers signature: 01.01.2010
Signatory:

D
DENTAURUM
GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany
Telefon +49 7231 / 803-0 · Fax +49 7231 / 803-295

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