Paraline

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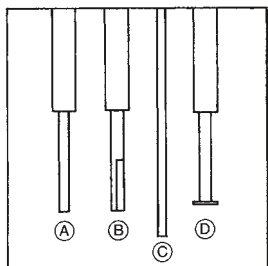
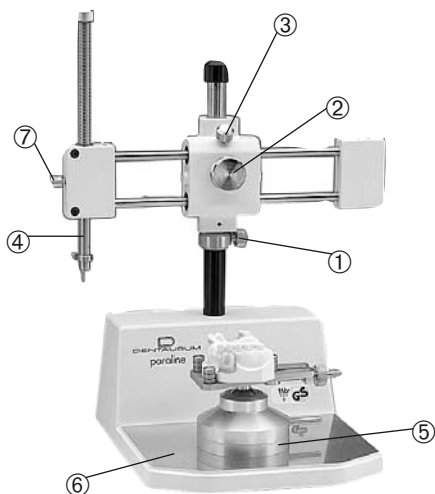
Instructions for use

Mode d'emploi

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Paraline

Vermessungsgerät

REF 094-290-00

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for partials

Original Kit

- 1 Paraline unit with surveying arm
- 1 Analyzing rod (A)
- 1 Graphite holder (B)
- 10 Replacement graphite markers (C)
- 4 Undercut gauges (D)
0.25/0.35/0.5/0.75 mm

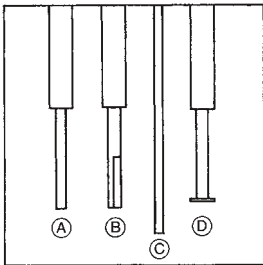
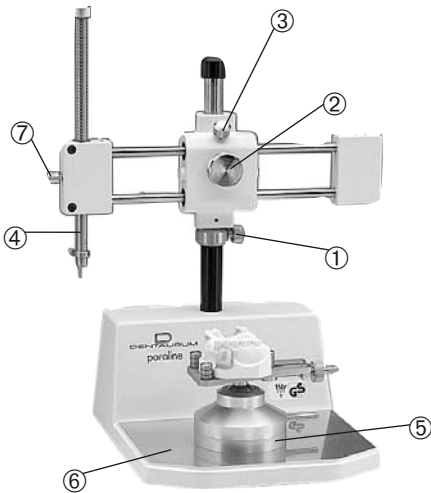
Recommended accessories

Model table (5) REF 086-340-00
Not included in Original Kit.

REF for those accessories

which are included in the Original Kit.

- 1 Analyzing rod REF 094-287-00 (A)
- 1 Graphite holder REF 094-288-00 (B)
- 10 Replacement graphite markers red REF 094-289-00 (C)
- 1 Undercut gauge REF 094-283-00
0.25 mm REF 094-284-00
0.35 mm REF 094-285-00
0.5 mm REF 094-286-00
0.75 mm

Instructions

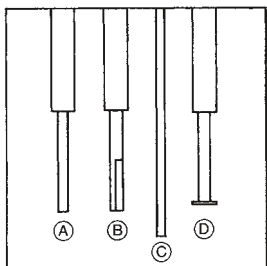
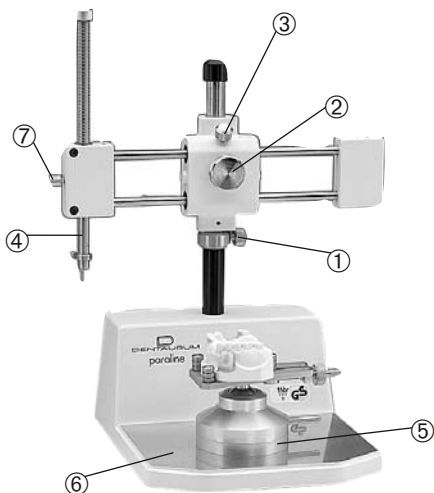
Place the Paraline on a level working area. Mount the master model on the model table already available or supplied with the unit. Loosen the knurled screw (2) and raise the sliding rods to the necessary level.

Fasten with retainer ring (1). Grip the instrument e. g. analyzing rod in the surveying hand-piece (4). Place the model table (5) in the most convenient position on the work surface (6). Loosen locking screw (3), so that the arms may be moved freely. By moving the surveying handpiece (4) vertically and horizontally, survey the model.

Locking the surveying arm (e. g. Attachments)

Rotate the surveying handpiece (4) forwards to the centre of the work surface and lock into position with screws (1), (2), and (3). Set the surveying arm to the required level and fasten using screw (7).

When ordering spare parts, please quote the unit number (type plate on rear side of unit).

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